


Flexible and automated testing for PBC

EliA M2 test



Primary biliary cholangitis (PBC)

Chronic inflammation of small intrahepatic bile ducts leading to impaired bile flow (cholestasis)^{1,2}

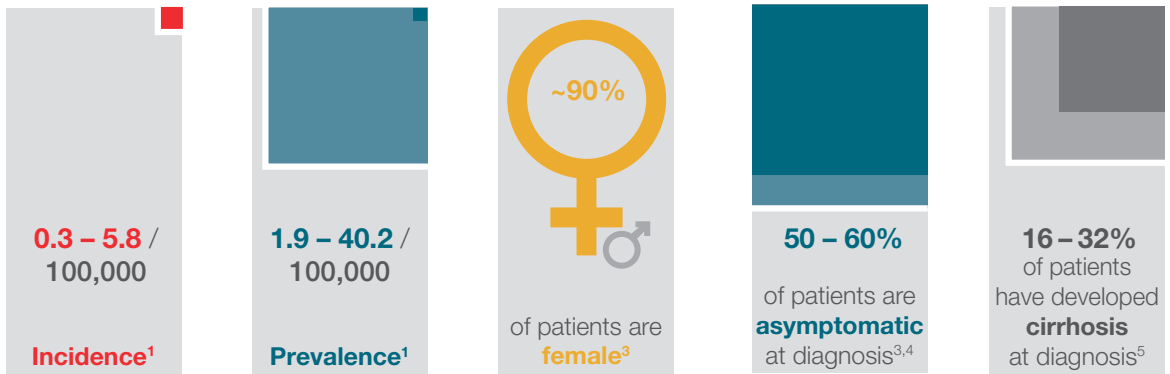

Identifying patients with PBC enables the clinician to improve patients' quality of life



Anti-mitochondrial antibodies (AMA)

- Measurement of AMA is an important aid in the diagnosis of PBC¹
- Often measured by immunofluorescence assay (IFA) that can be time-consuming and demanding in interpretation^{1,2}
- Positive AMA IFA should be confirmed with a more specific anti-M2 enzyme immunoassay (EIA)²
- It has been reported that in 24–100% of AMA IFA negative samples AMA can be measured with EIA⁶
- AMA against different mitochondrial proteins (M1 through M9) have been identified^{2,7}
- AMA against M2 (anti-M2) can be detected in $\geq 90\%$ of PBC patients¹
- M2 comprises several subunits with the pyruvate dehydrogenase complex representing the major antigen^{8,9}
- In a comparative study, anti-M2 EIA using a mixture of native and recombinant M2 antigen has been reported to have a higher sensitivity than anti-EIA using either native or recombinant M2 antigen alone¹⁰

Testing for anti-M2 antibodies



EliA™ M2 test – key features¹¹

- Antigen:** native pyruvate dehydrogenase complex from mitochondria plus recombinant human M2-antigen
- Controls:** EliA™ M2 Positive Control and EliA™ IgG/IgM/IgA Negative Control
- Automation:** available for Phadia™ 100, Phadia™ 200, Phadia™ 250 and Phadia™ 2500/5000 instruments
- Flexibility:** random access* testing with ready-to-use reagents to adjust to varying testing needs
- Efficiency:** easy add-on to EliA™ test portfolio by using the EliA™ IgG calibration method
- Cost savings:** interchangeable reagents with other EliA™ tests, e.g. EliA™ LKM-1 test and EliA™ CTD Screen^{12,13}

*random access testing available on Phadia 250 and Phadia 2500/5000 instruments



Clinical performance of EliA M2 test¹⁴

	Sensitivity [%]	Specificity [%]	PPV [%]	LR (+)
EliA M2 test (native M2 + recombinant M2)	81	98.1	95.3	42.5
Manufacturer 1 (native M2)	69	98.5	95.8	48.3
Manufacturer 2 (recombinant M2)	78	96.7	91.7	23.4

Study cohort: 100 PBC patients and 210 disease controls. PPV: Positive predictive value; LR(+): Positive likelihood ratio

- At a similar specificity, EliA M2 test had a higher sensitivity than the tests using only native or recombinant M2 antigen
- In a second study including 98 samples from PBC patients, EliA M2 test reported more positive samples than IFA (80 vs. 77)¹⁵

Technical data¹¹

Ordering information	Article No.	Package size	Cut-off			Short name
			negative	equivocal	positive	
EliA M2 Well	14-5649-01	4 x 12 wells	< 4 IU/ml	4–6 IU/ml	> 6 IU/ml	m2G

References

- European Association for the Study of the Liver. EASL Clinical Practice Guidelines: The diagnosis and management of patients with primary biliary cholangitis. *J Hepatol.* 2017;67(1):145-72.
- Sebode M, Weiler-Normann C, Liwinski T, Schramm C. Autoantibodies in Autoimmune Liver Disease-Clinical and Diagnostic Relevance. *Front Immunol.* 2018;9:609.
- Poupon R. Primary biliary cirrhosis: a 2010 update. *J Hepatol.* 2010;52(5):745-58.
- Chew M, Bowlus CL. Primary biliary cholangitis: Diagnosis and treatment. *Liver Research.* 2018;2(2):81-6.
- Prince MI, Chetwynd A, Craig WL, Metcalf JV, James OF. Asymptomatic primary biliary cirrhosis: clinical features, prognosis, and symptom progression in a large population based cohort. *Gut.* 2004;53(6):865-70.
- Ozaslan E, Efe C, Gokbulut Ozaslan N. The diagnosis of antimitochondrial antibody-negative primary biliary cholangitis. *Clin Res Hepatol Gastroenterol.* 2016;40(5):553-61.
- Han E, Jo SJ, Lee H, Choi AR, Lim J, Jung ES, et al. Clinical relevance of combined anti-mitochondrial M2 detection assays for primary biliary cirrhosis. *Clin Chim Acta.* 2017;464:113-7.
- Reshetnyak VI. Primary biliary cirrhosis: Clinical and laboratory criteria for its diagnosis. *World J Gastroenterol.* 2015;21(25):7683-708.
- Klein R. Primary biliary cirrhosis. In: The General Practice Guide to Autoimmune Diseases. Editors: Shoenfeld Y. and Meroni P. 2012:167-74.
- Klein R, Berg PA. Characterization of a new mitochondrial antigen-antibody system (M9/anti-M9) in patients with anti-M2 positive and anti-M2 negative primary biliary cirrhosis. *Clin Exp Immunol.* 1988;74(1):68-74.
- Directions for Use, EliA M2 250-5649-021 / UK
- Directions for Use, EliA LKM-1 250-6648-020 / UK
- Directions for Use, EliA CTD Screen 250-5596-022 / UK
- Thermo Fisher Scientific. Internal study.
- Thermo Fisher Scientific. Internal study.

Find out more at thermofisher.com/elia

ThermoFisher
SCIENTIFIC