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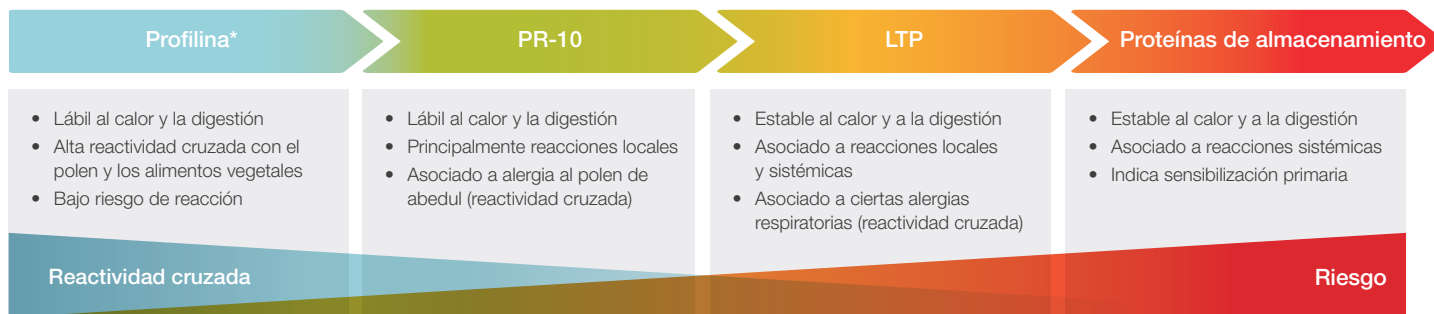
Nombres oficiales de los productos: ImmunoCAP Allergen f17, Hazelnut; ImmunoCAP Allergen f428, Allergen component rCor a 1 PR-10 Hazelnut; ImmunoCAP Allergen f425, Allergen component rCor a 8 Hazelnut; ImmunoCAP Allergen f440, Allergen component nCor a 9 Hazelnut; ImmunoCAP Allergen f439, Allergen component rCor a 14 Hazelnut; ImmunoCAP Allergen f256, Walnut; ImmunoCAP Allergen f441, Allergen component rJug r 1 Walnut; ImmunoCAP Allergen f442, Allergen component rJug r 3 LTP, Walnut; ImmunoCAP Allergen f18, Brazil nut; ImmunoCAP Allergen f354, Allergen component rBer e 1 Brazil nut; ImmunoCAP Allergen f202, Cashew nut; ImmunoCAP Allergen f443, Allergen component rAna o 3, Cashew nut; ImmunoCAP Allergen f13, Peanut; ImmunoCAP Allergen f422, Allergen component rAra h 1 Peanut; ImmunoCAP Allergen f423, Allergen component rAra h 2 Peanut; ImmunoCAP Allergen f424, Allergen component rAra h 3 Peanut; ImmunoCAP Allergen f447, Allergen component rAra h 6 Peanut; ImmunoCAP Allergen f352, Allergen component rAra h 8 PR-10, Peanut; ImmunoCAP Allergen f427, Allergen component rAra h 9 LTP, Peanut; ImmunoCAP Allergen f14, Soybean; ImmunoCAP Allergen f431, Allergen component nGly m 5 beta-conglycinin, Soy; ImmunoCAP Allergen f432, Allergen component nGly m 6 Glycinin, Soy; ImmunoCAP Allergen f353, Allergen component rGly m 4 PR-10, Soy; ImmunoCAP Allergen f10, Sesame seed; ImmunoCAP Allergen f449, Allergen Component rSes i 1 Sesame seed. ImmunoCAP Allergen f95, Peach; ImmunoCAP Allergen f419, Allergen component rPru p 1 PR-10, Peach; ImmunoCAP Allergen f420, Allergen component rPru p 3 LTP, Peach; ImmunoCAP Allergen f421, Allergen component rPru p 4 Profilin, Peach; ImmunoCAP Allergen f454, Allergen component rPru p 7 Peach; ImmunoCAP Allergen f4, Wheat; ImmunoCAP Allergen f416, Allergen component rTri a 19 Omega-5 Gliadin, Wheat; ImmunoCAP Allergen f433, Allergen component rTri a 14 LTP, Wheat; ImmunoCAP Allergen f49, Apple; ImmunoCAP Allergen f85, Celery; ImmunoCAP Allergen f435, Allergen component rMal d 3 LTP, Apple; ImmunoCAP Allergen f434, Allergen component rMal d 1 PR-10, Apple; ImmunoCAP Allergen f430, ImmunoCAP Allergen f84, Kiwi, Allergen component rAct d 8 PR-10, Kiwi; ImmunoCAP Allergen f417, Allergen component rApi g 1 PR-10, Celery; ImmunoCAP Allergen f98, Gliadin

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© 2025 Thermo Fisher Scientific Inc. Todos los derechos reservados. Todas las marcas comerciales son propiedad de Thermo Fisher Scientific y de sus filiales, salvo que se especifique lo contrario. Fabricante legal: Phadia AB (miembro de Thermo Fisher Scientific). 453351.AL.EU4.ES.V1.25